



# BD AutoShield Duo™ Safety Pen Needle with dual automatic protective shields

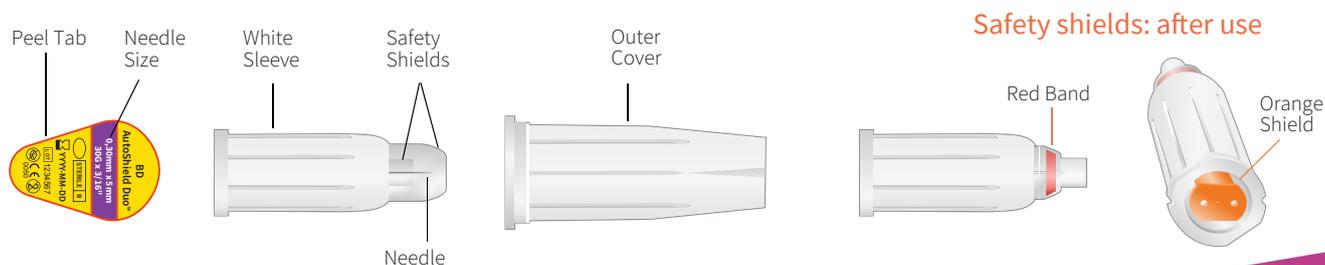


Safety shields: after use


## How to inject using a BD AutoShield Duo™ Safety Pen Needle

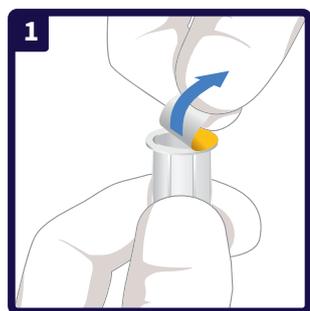


1 Pull the peel tab off the pen needle.

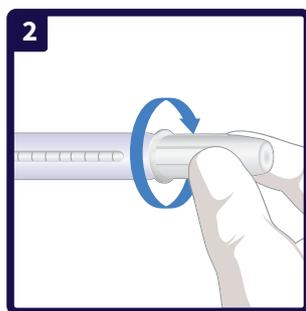

2 Holding the outer cover, screw the pen needle straight onto the pen.

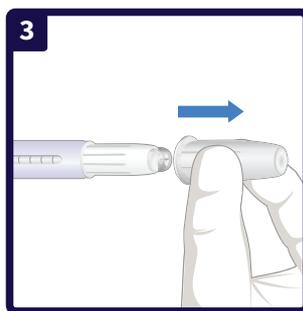


3 Remove the outer cover.

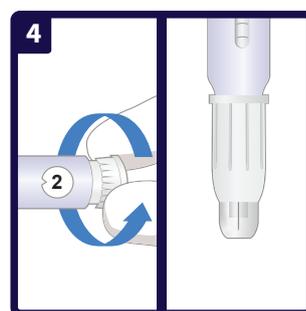


4 Dial 2 units for priming the pen. Hold the pen upright with the needle at eye level.\*




5 Dispense the 2 units. Check pen needle is open and working. You may flick the pen to remove the priming droplets.

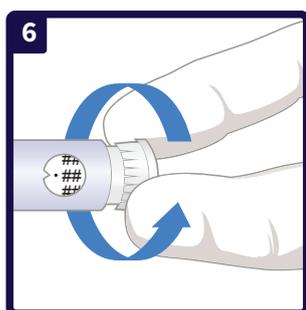


6 Dial the prescribed dose.

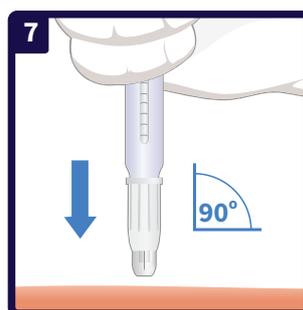


7 Use one hand to brace the skin and the other to inject at a 90-degree angle. You will hear a click when the needle shield makes contact with the skin.<sup>1†</sup>

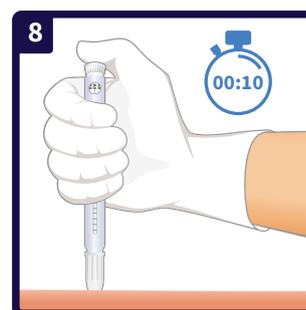

8 Push the dose button and leave the needle in the skin for 10 seconds.<sup>1</sup>

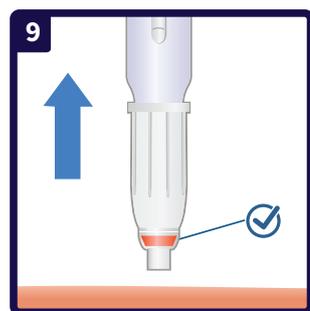


9 Withdraw the pen needle from the skin. A red indicator band appears inside the clear safety shield.

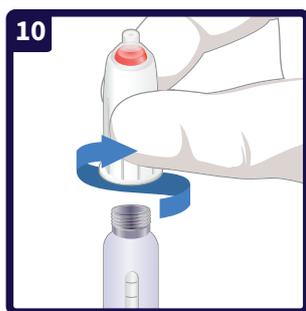

10 Hold the white sleeve of the pen needle and twist it off.



11 The pen connection end of the pen needle is now protected with an orange shield.

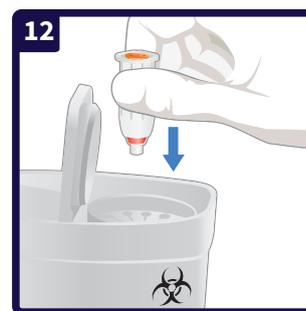

12 Dispose of the pen needle in a sharps container.

\* Priming should be completed in accordance with the steps recommended by the pen manufacturer. It is important to prime the needle successfully before administering the injection.

† Patients 6 years old and younger as well as very thin adults may require a pinch-up.

<sup>1</sup>Frid AH, Kreugel G, Grassi G, et al. New insulin delivery recommendations. Mayo Clin Proc. 2016;91(9):1231-1255.

[embecta.com](http://embecta.com)

embecta, formerly part of BD. BD is the manufacturer of the advertised products. embecta and the embecta logo are trademarks of Embecta Corp. BD and the BD Logo are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners.

© 2023 Embecta Corp. All rights reserved. BD-110330

embecta™